


# Fentanyl Drug Test

## Test Procedure




### 1 What do you need

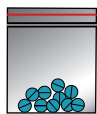

**DRUGS**



**CAP and WATER**

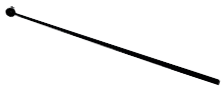

**MICROSCOOP (5-10 mg)**



**FENTANYL TEST STRIP**

### 2 Crush and shake

0.1



**Crush**



0.2



**Powder**



0.3




**Shake**

=



**Rock**

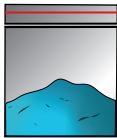

**Powder**

### 3 Measure and dilute

**ALL DRUGS**  
**5 mg**

=

0.1



=1 tsp

=1 cap



0.2



0.3



**1 Scoop = 5 mg**



**NOTE:** A single scoop holds the recommended quantity of drug to be diluted in 1 tsp (5mL) of water prior to testing with BTNX's Rapid Response Harm Reduction Drug Test Strips